


# How to perform MCG test

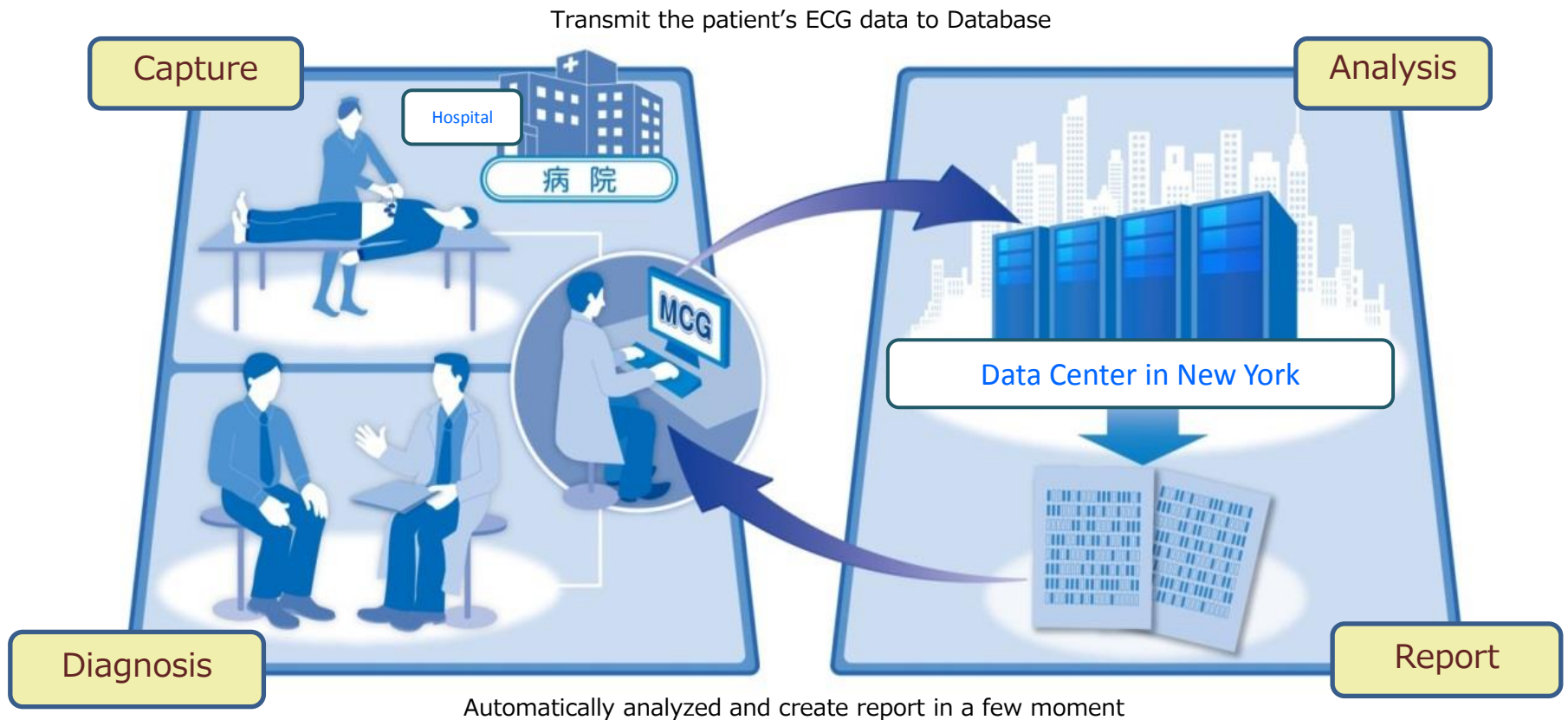

MCG test is very simple.

- (1) Capture ECG data like ordinal ECG for 5-10 minutes.
- (2) Send the data to the data center via internet.
- (3) Data is analyzed automatically, and receive a report.

# MCG machine

MCG  
Machine



MCG V5  
electrode



MCG Clips

MCG uses 2 leads (V5 and II) for testing.

For II lead, 4 clips (arms and legs ) should be attached.

For V5 electrode should be attached to the breast.

MCG  
patient cable


# MCG test



① Input patient data



② Attach V5 lead electrode

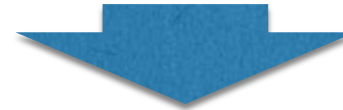

④ Start recording 5-10 minutes.



③ Attach II lead clamps

After finish, send the recorded data to our server.

# Report is ready for download immediately after automatically analyzed

## Report downloading



PREMIER HEART


ログイン

メールアドレス   
パスワード

ログイン

This site officially supports the Mozilla Firefox and Google Chrome web browsers. Javascript and session cookies must be enabled.

① Access and login to our website from your PC

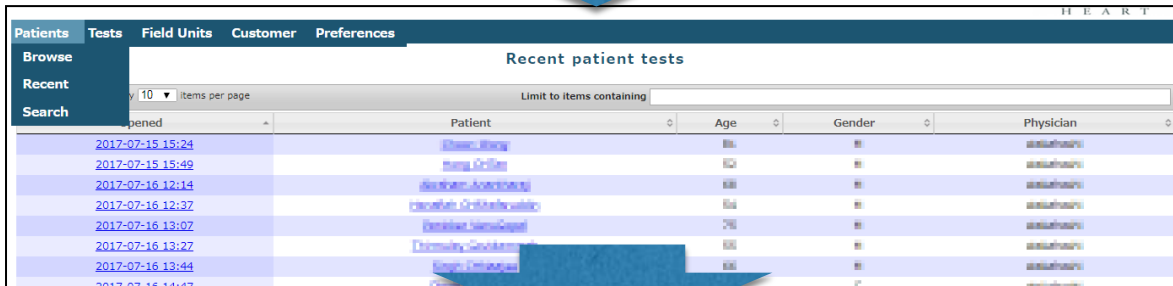

PREMIER HEART

Patients Tests Field Units Customer Preferences

Browse Recent Search


Recent patient tests

Limit to items containing

Opened	Patient	Age	Gender	Physician
2017-07-15 15:24	<a href="#">Shiro, Shiro</a>	65	M	<a href="#">Saitoh, Shiro</a>
2017-07-15 15:49	<a href="#">Hiro, Hiro</a>	65	M	<a href="#">Saitoh, Shiro</a>
2017-07-16 12:14	<a href="#">Ishida, Akiko</a>	65	F	<a href="#">Saitoh, Shiro</a>
2017-07-16 12:37	<a href="#">Ishida, Akiko</a>	65	F	<a href="#">Saitoh, Shiro</a>
2017-07-16 13:07	<a href="#">Ishida, Akiko</a>	70	F	<a href="#">Saitoh, Shiro</a>
2017-07-16 13:27	<a href="#">Ishida, Akiko</a>	65	F	<a href="#">Saitoh, Shiro</a>
2017-07-16 13:44	<a href="#">Ishida, Akiko</a>	65	F	<a href="#">Saitoh, Shiro</a>

② Select the patient

③ See the result

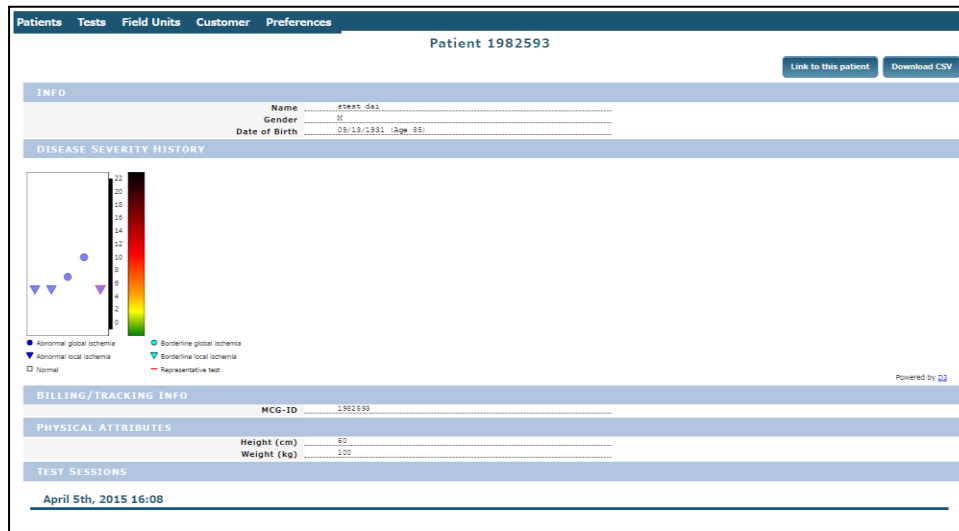

PREMIER HEART

Patients Tests Field Units Customer Preferences

Patient 1982593

Link to this patient Download CSV

INFO

Name: [Shiro, Shiro](#)  
Gender: M  
Date of Birth: 08/18/1951 (Age: 65)

DISEASE SEVERITY HISTORY

BILLING/TRACKING INFO

MCG-ID: 1982593


PHYSICAL ATTRIBUTES

Height (cm): 65  
Weight (kg): 55

TEST SESSIONS

April 5th, 2015 16:08

④ Download report

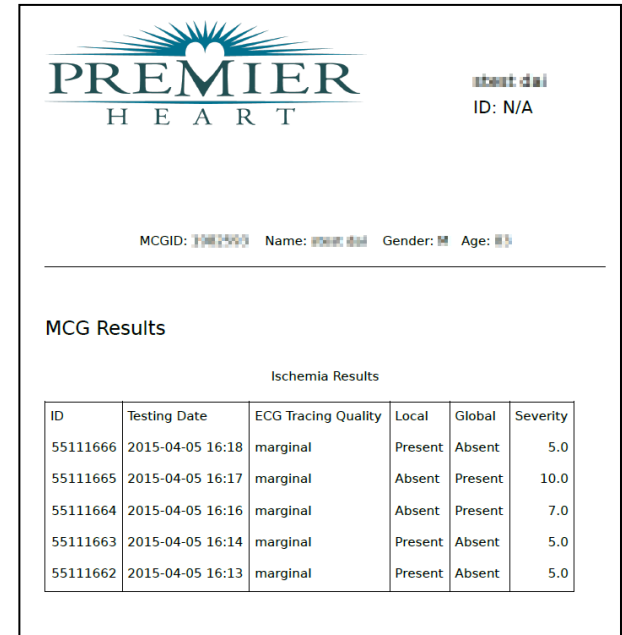

PREMIER HEART

test: dai  
ID: N/A

MCGID: 1982593 Name: [Shiro, Shiro](#) Gender: M Age: 65

MCG Results

Ischemia Results

ID	Testing Date	ECG Tracing Quality	Local	Global	Severity
55111666	2015-04-05 16:18	marginal	Present	Absent	5.0
55111665	2015-04-05 16:17	marginal	Absent	Present	10.0
55111664	2015-04-05 16:16	marginal	Absent	Present	7.0
55111663	2015-04-05 16:14	marginal	Present	Absent	5.0
55111662	2015-04-05 16:13	marginal	Present	Absent	5.0